sigvaris

www.sigvaris.com/videos

DE: Materialverzeichnis:
Thermoplastisches Elastomer
DND: Füllung: 95 % Wasser, 5 % Polyacrylamid / **CONE:** Polyethylen
EN: List of Materials:
DND: Thermoplastisch elastomer
Contents: 95 % water, 5 % polyacrylamide / **CONE:** Polyethylene
FR: Liste des matériaux:
DND: Elastomère thermoplastique
Remplissage: 95 % eau, 5 % polyacrylamide / **CONE:** Polyéthylène
IT: Elenco dei materiali:
DND: elastomero termoplastico
Contenuto: 95% acqua, 5% poliacrilamide / **CONE:** Polietilene
DA: Liste over materialer:
DND: Termoplastisk elastomer
Påfyldning: 95 % vand, 5 % polyakrylamid / **CONE:** Polyethylen
NL: Toegepaste materialen:
DND: thermoplastisch elastomeer
Vulling: 95 % water, 5 % polyacrylamide / **CONE:** Polyethyleen
PL: Wykaz materiałów:
DND: elastomer termoplastyczny
Skład: 95 % woda, 5 % poliakrylamid / **CONE:** polietylen
NO: Materialeoversikt:
DND: Termoplastisk elastomer
Fylling: 95 % vann, 5 % polyakrylamid / **CONE:** polyetylen
FI: Materiaalit:
DND: termoplastinen elastomeeri
Täyte: ves 95 %, polyakryylamidi 5 % / **CONE:** polyeteeliini
SV: Material:
DND: Termoplastisk elastomer
Fyllning: 95 % vatten, 5 % polyakrylamid. **CONE:** Polyetylen

■ SIGVARIS AG, Gröblistrasse 8, 9014 St. Gallen, Switzerland
Phone +41 71 272 40 00
SIGVARIS GmbH, Dr.-Karl-Lenz-Strasse 35, 87700 Memmingen, Germany
Phone +49 8331 757 0
SIGVARIS, medicinske kompresjonsstrømper Geselschaft m.b.H., Hietzing Hauptstrasse 22/2/3,
1150 Vienna, Austria
Phone +43 1 877 69 12
SIGVARIS S.r.l., Via Galileo Galilei 2, 31057 Silea (TV), Italy
Phone +39 422 052 750
SIGVARIS S.A., ul. Povidzka 50, Głogów Małe, 62-300 Września, Poland
Phone +48 61 43 88 200
SIGVARIS (Australia) Pty. Ltd., 81/53 Stanley Street, West Melbourne VIC 3003, Australia
Phone 1300 SIGVARIS (1300 744 827)
SIGVARIS (SHANGHAI) TRADING CO LTD, Pudong New District, Unit 207, Building 1, Lane 677, Wuxiang
Road, Shanghai 201204, China
Phone +86 21 6882 3908
SIGVARIS SAS, 21, Suis d'Andrézieux, Rue Barthélemy Thimonnier, 42170 St-Just-St-Rambert, France
Phone +33 4 77 36 08 90
SIGVARIS Britain LTD, 1 Imperial Court, Magellan Close, Andover, SPO SNT, Great Britain
Phone +44 1264 326 666

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EN: **DOFF N' DONNER PRODUCTS consisting of:**
A: DND B: CONE (sold separately)
Indications: • Impaired ability to don and doff the compression stockings if the function of the arms, the hands, or the fingers is impaired and in the case of reduced strength (e.g. in the case of joint diseases, neurological diseases, consequences of injury, deformities) • For donning and doffing compression stockings without assistance
Contraindications: No known contraindications.
Suitable for the following models:
A-D Calf stocking, open/closed toe, with or without grip top
A-G Thigh stocking, open/closed toe, with or without grip top

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EN: **Intended Purpose:** Facilitates donning and doffing of medical compression stockings.
Description of Function: The DOFF N' DONNER (DND) is a soft device that allows you to quickly and painlessly slide medical compression stockings over an arm or a leg. The CONE simplifies the rolling of a compression stocking onto the DND. With the aid of the CONE, the medical compression stocking can also be easily unrolled from the DND.
Target Group: Patients and nursing/care staff.

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
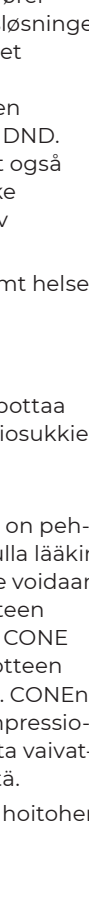
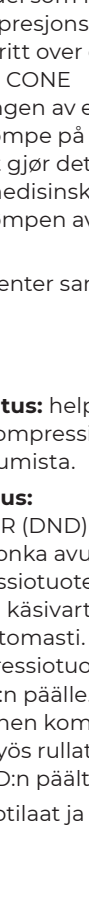
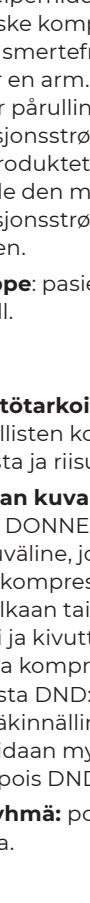
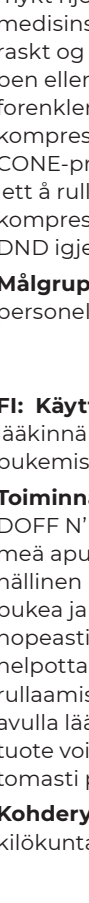
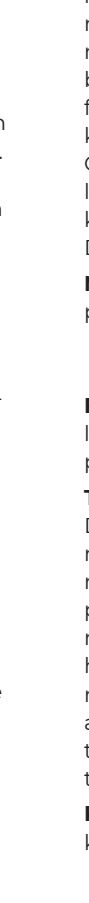
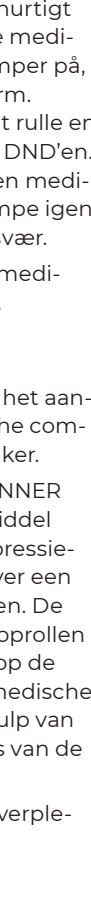
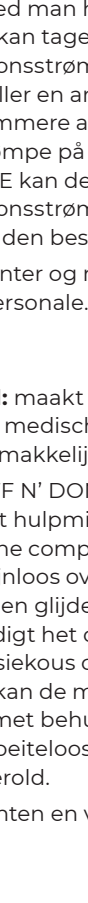
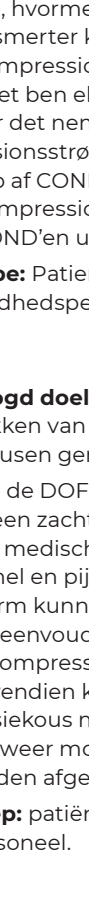
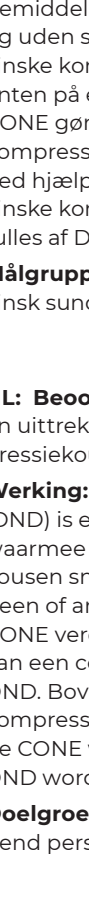
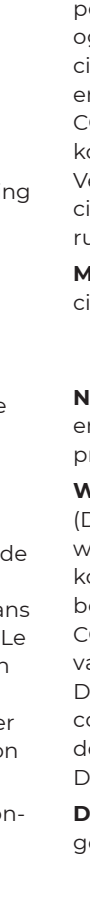
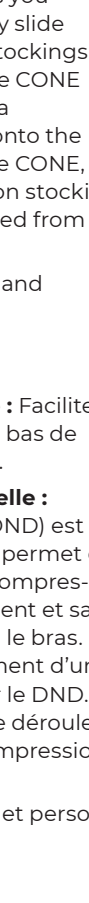
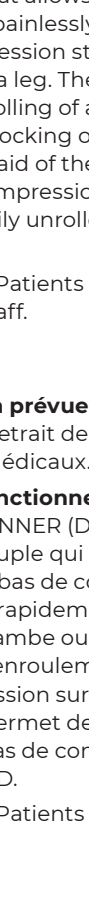
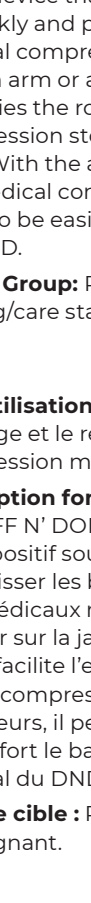
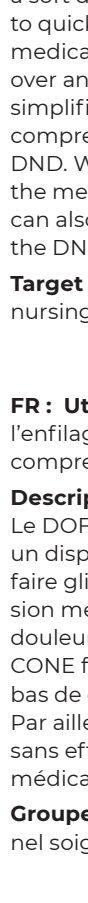
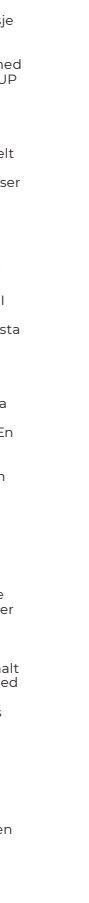
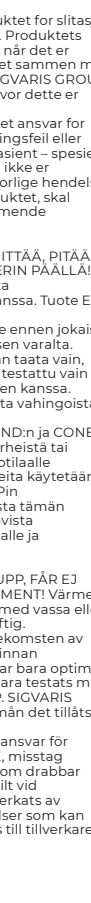
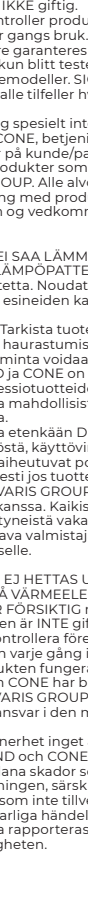
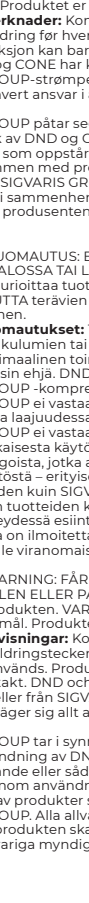
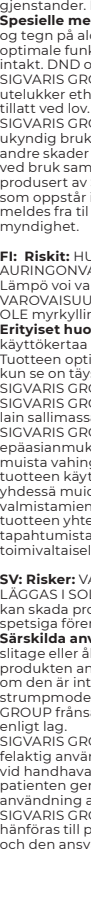
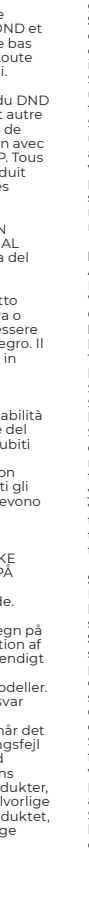
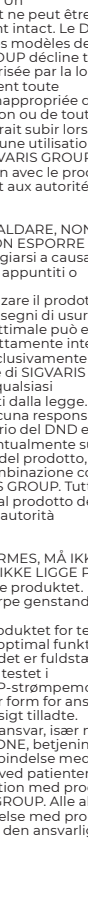
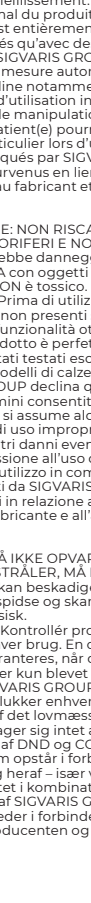
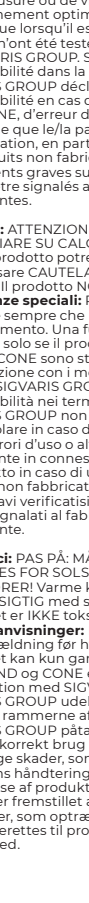
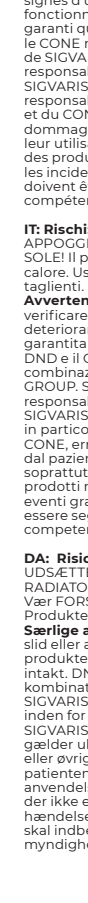
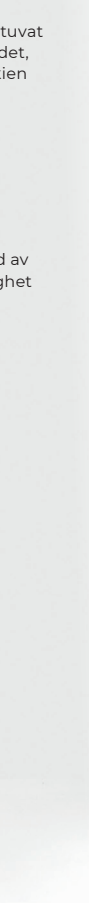
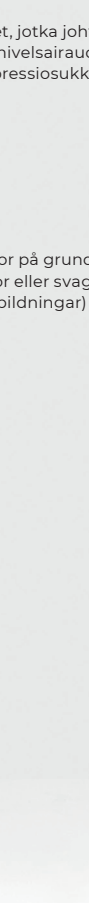
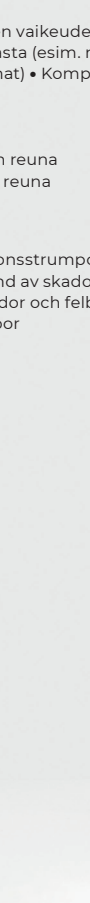
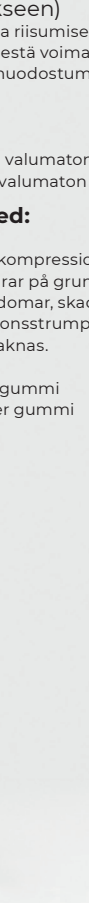
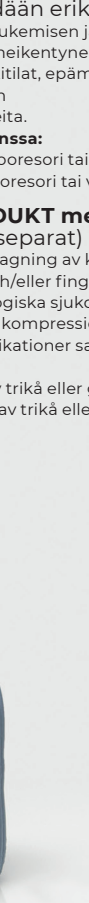
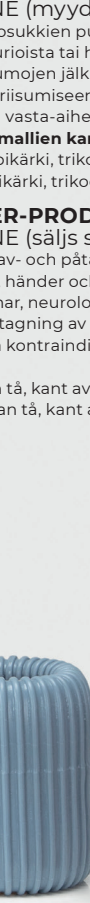
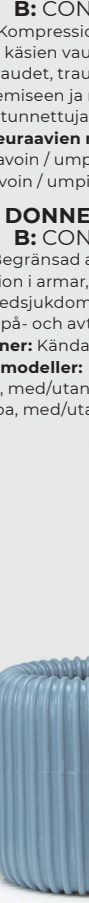
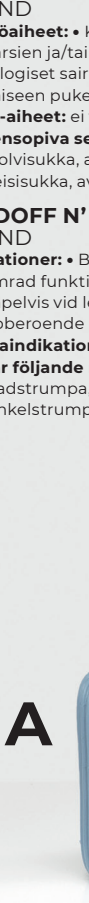
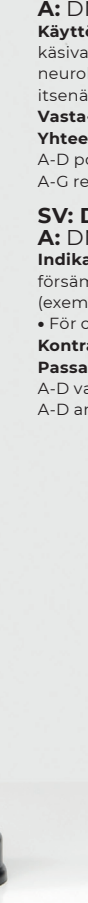
EN: Donning
Clamp the CONE to a smooth surface (press the lever down, CONE adheres firmly).

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SIGVARIS and SIGVARIS GROUP are registered trademarks of SIGVARIS AG,
© 2022 Copyright by SIGVARIS AG, CH-9014 St. Gallen/Switzerland